


BioHermes®

# IGFBP-1/fFN

Point-of Care Test (POCT) IGFBP-1/fFN  
Obstetrical Diagnosis Expert

JOINTLY TESTING MAKES  
YOUR PATIENTS MORE SECURE

National Patent: ZL201020565061X

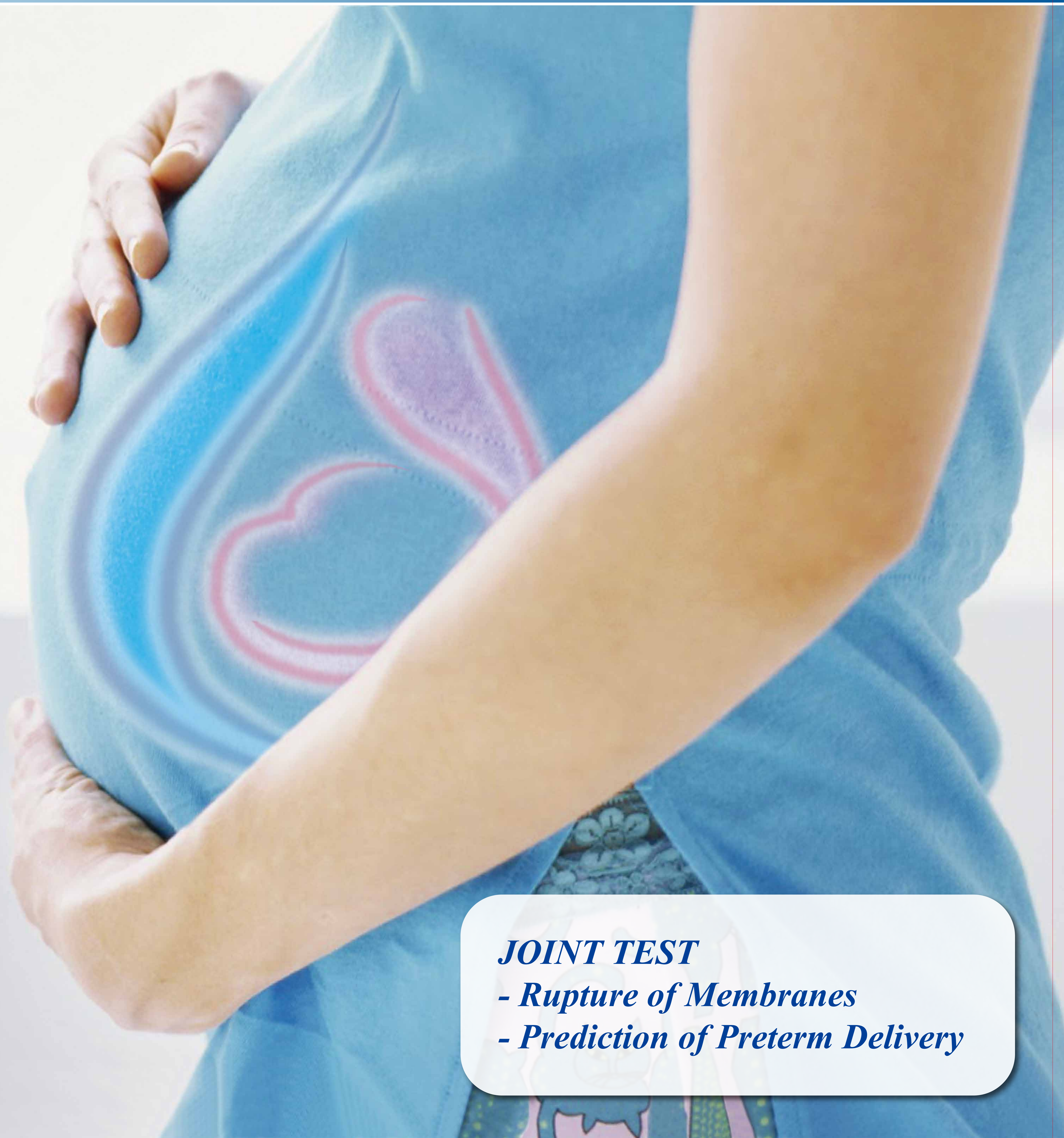

## JOINT TEST

- Rupture of Membranes
- Prediction of Preterm Delivery

- ★ Scientific diagnosis of Rupture of Membranes and Prediction of Preterm Delivery by one sampling, reducing the risk;
- ★ High specificity and sensitivity;
- ★ SIMPLE, FAST, NON-INVASIVE;

T1: IGFBP-1  
T2: fFN

Result Interpretation

**POSITIVE**  
Red reaction zones appear on C, T1 and / or T2.

**NEGATIVE**  
Only one red reaction zone appears on C, it means IGFBP-1 <12.5µg/l and fFN < 50µg/l

NEGATIVE POSITIVE

**NEGATIVE:** 1. No red reaction zone appears on T1 and T2: no matter with or without aura symptoms of Preterm Delivery, basically ruling out the possibility of delivery in two weeks.

**POSITIVE:** 1. Red reaction zones appear on T2: the risk of Preterm Delivery increased, the possibility of pregnant women giving birth within 7 days is more than 50%;  
2. Red reaction zones appear on T1: occult membranes rupturing may occur, the risk of typical premature rupture of membranes is increased;  
3. Red reaction zones appear on T1 and T2, the risk of Rupture of Membranes and Prediction of Preterm Delivery is super.

